

EVALUATION OF GRADIENT PET SEGMENTATION FOR TOTAL LESION GLYCOLYSIS COMPARED TO THRESHOLDS AND MANUAL CONTOURING

A.S. Nelson¹, M.Werner-Wasik², W. Choi³, Y.Arai⁴, P.F. Faulhaber⁵, N. Ohri², P. Kang³, F. Almeida⁶, J.W. Piper¹, S. Pirozzi¹, A.D. Nelson¹

¹ MIM Software Inc., Cleveland, OH • ² Thomas Jefferson University Hospital, Philadelphia, PA • ³ Beth Israel Medical Center, New York, NY ⁴ University of Pittsburgh Medical Center Health Systems, Pittsburgh, PA • ⁵ University Hospitals Case Medical Center, Cleveland, OH • ⁶ University of Arizona Health System, Tuscon, AZ

Purpose & Objectives


Assessing response to therapy or prognosis using SUVmax has a number of limitations including the sensitivity of SUVmax to noise. Total lesion glycolysis (TLG) is less sensitive to noise and incorporates both size and activity (SUVmean x volume) into one measure. TLG has been shown to have value in both determining prognosis and response to therapy¹⁻⁴. TLG has taken on additional importance as a parameter that can be measured as part of the PERCIST 1.0 guidelines⁵.

Accuracy of PET segmentation is important for TLG since it utilizes volume. Previously we demonstrated the superior volumetric accuracy of a gradient PET segmentation method (GRAD) compared to SUV thresholds (THRESH) in sphere phantoms⁶ and clinical lung cancer patients using maximal pathological diameter for comparison⁷ and compared to THRESH and manual contouring (MC) using realistic Monte Carlo simulated PET scans of the thorax⁸. In this study we evaluated the effect of segmentation accuracy on TLG.

Methods & Materials

Thirty-one lung tumors of varying size, shape, and location were segmented by 7 clinicians on 25 realistic digital PET scans of the thorax⁹. GRAD, THRESH and MC methods were used. GRAD identifies tumor edges based on a change in count levels at the tumor border. THRESH was performed using 25-50% of maximum counts at 5% increments. Accuracy and bias were measured by calculating the mean absolute % error and mean % errors respectively (abs%error and %error) for TLG using all methods.

Figure 1 Spatial Derivative

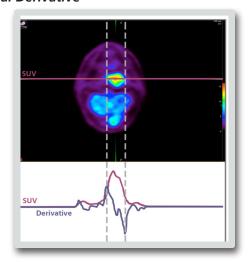

Results


GRAD was the most accurate technique with abs%error of 6.1 (10.3 SD). Both 25% THRESH, the most accurate threshold, and MC were significantly less accurate with abs%error of 10.2 (19.1) and 14.5 (19.2) respectively (p < 0.0013). 25% THRESH had the smallest bias with %error of -1.1 (21.6) followed by GRAD with -3.8 (11.4), however, the difference was not significant (p = 0.13).

Table 1

	MC	GRAD	25%	30%	35%	40%	50%	Mean
Average Absolute % Error	14.50	6.15	10.23	13.14	16.38	22.50	37.18	17.5
Standard Deviation	19.16	10.32	19.09	15.87	11.12	15.18	14.06	
p-values								
p-values PET vs 25% THR	ESH	GRAD	vs MC					

Figure 2

Digital PET Phantom - NSCLC

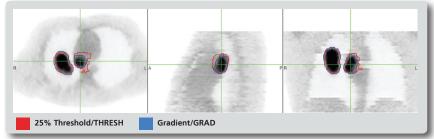

For cases where the tumor is in the center of lung (i.e. high source-to-background) and fairly homogeneous, 25% THRESH performs fairly well, however, in the mediastinum with lower source-to-background 25% THRESH performs poorly. GRAD produces more accurate segmentations in both of these scenarios.

Conclusions

GRAD resulted in significantly more accurate TLG calculations than the other contouring methods. GRAD has the potential to play an important role in both determining prognosis and assessing response to therapy.

References

University Hospitals

Case Medical Center

- 1. La TH et al. Metabolic Tumor Volume Predicts for Recurrence and Death in Head-and-Neck Cancer. IJROBP 2009; 74(5):1335-1341.
- 2. Lee P et al. Metabolic Tumor Volume Predicts Overall Survival in Lung Cancer Patients Treated Definitively. IJROBP Sept 2008; 72(1):S462.
- 3. Deeb E et al. A novel methodology for assessment of NHL B-Cell lymphoma by comparing tumor metabolic index to tumor volume. J Nucl Med. 2007; 48(Supplement 2): 450P.
- Muthukrishnan A et al. F-18 FDG PET-CT tumor metabolic index analysis as an early and accurate biomarker in management of NHL B-cell lymphoma. J Nucl Med. 2007;48(Supplement 2):352P.
- 5. Wahl R et al. From RECIST to PERCIST: Evolving considerations for PET response criteria in Solid Tumors. J Nucl Med 2009; 50:1225-1505.
- 6. Shen G et al. PET Tumor Segmentation: Comparison of Gradient-Based Algorithm to Constant Threshold Algorithm. Medical Physics June 2007;34(6):2395.
- 7. Fogh S, Karancke J, Nelson AS, McCue P, Axelrod R, Werner-Wasik W. Pathologic Correlation of PET-CT Based Auto-contouring for Radiation Planning in Lung Cancer. Presented at World Conference on Lung Cancer Meeting in 2009.
- Werner-Wasik M et al. What is the best way to contour lung tumors on PET scans? Multiobserver validation of a gradient-based method using a NSCLC digital PET phantom. IJROBP 2011. Article in Press.
- 9. Aristophanous M, Penney BC, Pelizzari CA. The development and testing of a digital PET phantom for the evaluation of tumor volume segmentation techniques. Med Phys. 2008;35:3331-3342



